Happy Intim® Peel

LANGUAGE:

ENG.....	2
ESP.....	3
BUL.....	5
HRV.....	7
CES.....	8
DAN.....	9
NLD.....	10
EST.....	12
DEU.....	13
GRE.....	15
HUN.....	17
ITA.....	19
LIT.....	22
POL.....	23
POR.....	25
RON.....	27
SLO.....	28
SLV.....	29
SWE.....	30

ENG

The Medical Device Class IIa **HAPPY intim® Peel** is a peeling of TCA 15% w/w less painful compared to a classic TCA peeling.

PRELIMINARY INFORMATION

TCA: Trichloro acetic acid. Contains 3% phenol.

This product is for the exclusive use by medical specialists in plastic aesthetic or restorative surgery, by specialists in medical and surgical dermatology and venereology, and M.D.s with postgraduate training in aesthetics.

PRELIMINARY PRECAUTIONS

Before starting the use of **HAPPY intim® Peel** read carefully this instruction for use. The information contained in the records is given as an indication, it helps the doctor to select the proper peeling and the best way of applying it and, consequently, the doctor retains full responsibility for the decision and the act committed. The deeper is the peeling and the higher is the risk of complications. **HAPPY intim® Peel** contains phenol, the physician must consider the possibility of toxicity of phenol during its use. Avoid contact with the eyes: phenol can cause a possibly definitive clouding of the cornea, with partial or total, temporary or permanent loss of vision.

PRESENTATIONS

· Kit **HAPPY intim® Peel** 2 peelings.

KIT DESCRIPTION

The number of elements varies and it is adapted to the number of peelings provided.

- **HAPPY intim® Base:** Solution for diluting TCA to a concentration of a 15% w/w. 2 peelings: 1 vial of 10 mL containing 4 mL
- **HAPPY intim® Active:** Solution containing TCA 50% w/w. 2 peelings: 1 vial of 5 mL containing 1,5 mL
- **Divers:** 1 instructions for use, 2 pairs of traceability stickers.

INDICATIONS

HAPPY intim® Peel is primarily intended for a peeling until the basal layer of the epidermis, but it can reach the Grenz zone or the papillary dermis, depending on the application process and the number of coats applied. The indications are: photo-aging and its consequences, acne and post acne, pigmentation problems (melasma, chloasma, post inflammatory hyperpigmentation), ichthyosis, keratoses and lentigine.

CONTRAINDICATIONS

HAPPY intim® Peel is intended for the treatment of intim zones skin. Application of more than 2,7 mL of peeling solution could induce the usual complications of phenol peels. In particular, do not apply **HAPPY intim® Peel** on patients with previous or on-going treatment that could increase the skin permeability.

PRE-PEELING SKIN PREPARATION

Cleanse the skin with the foam "Skin Tech® Cleanser" (no alcohol or acetone); give a classical herpes prevention in case of previous herpes infection. The selection of an eventual pre-peeling preparation must be done taking into account the personal specificities. In any case, the patient should apply the cream "HAPPY intim® Lightening Cream" twice a day, during 15 days prior to the peeling.

PREPARATION OF THE PEELING SOLUTION

Syringe out of 1,5 mL of **HAPPY intim® Active** and deposit it into the vial of **HAPPY intim® Base** and mix the vial to obtain the peeling solution **HAPPY intim® Peel**.

APPLICATION OF PEELING

HAPPY intim® Peel is usually applied with circular movements, using cotton buds (called "the applicators").

APPLICATION OF THE FIRST LAYER

Divide the zone to be treated in small parts, example: monte venus 2 parts, internal part of legs 3 parts..) Soak the applicators into the peeling solution, spin them dry on the edge of the container in order to eliminate an eventual excess of solution and apply on the first area using regular circular movements; repeat this process for each area. Let this first layer dry completely and check "the result". "The result" is not a volumetric notion (millilitres) or a temporal concept (minutes) but rather a clinical sign: frosting points sign a peeling until the basal layer and frosting clouds sign a peeling until the Grenz zone. A uniform pink-white frosting signs a peeling until the papillary dermis.

APPLICATION OF POSSIBLE NEXT COATS

Apply the peeling solution using circular movements. Let this second layer dry and check "the result" (see above).

PREDICTABLE EFFECTS DURING THE FIRST WEEK POST PEELING

Dry skin begins to peel on the third day (sometimes later in case of first peeling) and finishes until the sixth day. The desquamation intensity depends on the depth of the peeling.

- **Depth 2/7 (intra-epidermal):** desquamation almost invisible.

- **Depth 3/7 (basal layer):** desquamation 3 days like sunburn.

- **Depth 4/7 (Grenz zone):** the skin may darken in peeling zones 4/7 and it results in a more visible peeling.

DAILY CARE

Starting from the day following any peeling, sun protection: Skin Tech® Melablock-HSP SPF 30 or 50+ is required in areas exposed to sunlight, throughout the treatment and until the end of the second week after the last peeling (applications at 9am, 1pm, 3pm). From the day after the first peeling, the patient shall apply once or twice a day **HAPPY intim® Lightening Cream** Generally, if any cream produces a sensation of intense heat, suspend the application for a few days.

REPETITION OF PEELINGS

HAPPY intim® Peel is repeated 2 times in general, according to the decision of the doctor, one session a week. 2 sessions in total for each treatment that may be repeated after at least 6 weeks of the skin's rest period.

PRECAUTIONS

Avoid direct sun exposure during the treatment periods. Do not apply on the damaged or infected skin or after microneedling or dermabrasion, or on open wounds or on recent scars. Do not rub or pull the desquamated skin. Wash hands before and after use. Do not inhale; do not allow the peeling solution to enter the body by any means whatsoever, other than the application on the skin. Do not use on wet skin (dilution of active ingredients) or dirty skin (the organic substances present on the skin could reduce the action of the acids). Avoid contact with eyes: In case of contact with eyes, an intense burning sensation appears: Rinse immediately with saline solution until the burning sensation disappears. Phenol can induce corneal opacification. Consult with a specialist if the solution comes in contact with eyes. If swallowed, take appropriate protective measures, consult a physician immediately and show the container or label. If necessary, contact with a toxicological center nearest you. Store between +5 °C and +35 °C. Do not use if the packaging (box or bottle) has been damaged. Keep out of the reach of children. Not suitable for preteens.

POSSIBLE SECONDARY EFFECTS

Patient could suffer side effects during and after the peeling such as:

- (minor, disappear in 24h) burning sensation/Itching/Erythema/Swelling/
- (not common, please contact the doctor who treated you in any case) Allergic reaction/Ulceration/Hyper or hypo-pigmentation/Scarring/Secondary infection/Cardiac arrhythmia...

The doctor is requested to report any notable side effect related to the medical device **HAPPY intim® Peel** to Skin Tech Pharma Group SL. through the website www.skintechpharmagroup.com, by e-mail at info@skintechpharmagroup.com or by post.

DISPOSAL

Comply with local, national, and European regulations for the disposal of wastes and toxic chemical hazards of contaminated wastes.

	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C - 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restilize
	Sterilized using steam or dry heat
	Temperature limitation 2°C - 35°C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not sterilize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation 2°C - 35°C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not sterilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C - 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restitilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C - 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restitilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C — 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries - Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Use by
	Do not use if package is damaged
	Do not reuse
	Do not restirize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number