RRS[®] HA SKIN RELAX WITH BONTA 568[®]

LANGUAGE:

ENG.....	2
SPA.....	3
HRV.....	4
CES.....	5
DAN.....	6
NLD.....	7
EST.....	8
GRE.....	9
HUN.....	10
LAV.....	11
LIT.....	12
POL.....	13
RON.....	14
SLO.....	15
SLV.....	16
SWE.....	17

ENG

DESCRIPTION

RRS® HA SKIN RELAX WITH BONTA 568® is a sterile medical device, class III, made in conformity with Directive 93/42/EEC, dermal resorbable implant that contains a non-cross-linked hyaluronic acid associated to protective buffer that helps to prolong the effect of the dermal implant with skin relaxing action. Hyaluronic acid is from biotechnological non animal origin. A medical device only can be injected by a legally authorised physician in a legally authorized clinic.

PRESENTATION

Vials containing 3 ml (0,10 Fl.oz.)

Boxes of 2 vials, 6 vials, 12 vials.

COMPOSITION

Sodium Hyaluronate	3 mg / 3 ml
Protective buffer	0,9 mg / 3 ml

INDICATIONS

RRS® HA SKIN RELAX WITH BONTA 568® Dermal implant that contains a non reticulated hyaluronic acid solubilized in a protective buffer that induces a dermal relaxation in order to allow better diffusion of hyaluronic acid into the Dermis where the latter suffers from photo aging associated with excessive skin tension.

HOW TO USE

RRS® HA SKIN RELAX WITH BONTA 568® is injected in the dermis (0.2 - 0.6 mm), using a 32 G needle. The physician will use the method which best suits, and among others may be with: as superficial as possible multiple longitudinal retro-tracing injections, fanning outwards, criss-cross technique, fish-bone technique. Automated techniques for injections like mesotherapeutic guns may also be used. Any injection may leave small ecchymoses points or even haematomas, mainly on patients taking acetylsalicylic acid, anti-inflammatory or taking medication delaying coagulation. Sensitive skins may benefit from application of an anaesthetic cream prior to the treatment; said cream may cause redness or skin hypersensitivity.

Before implantation, the physician should inform the patient about the possibilities of side effects (pain, redness, ecchymoses, stinging sensations, swelling, local inflammation, usually disappearing in 24 hours) and check that he is healthy. **RRS® HA SKIN RELAX WITH BONTA 568®** must be used under appropriate aseptic conditions in an authorised clinic on healthy, disinfected, non-inflamed skin. Non aseptic use could induce infections. After the treatment, stick the traceability label on the patient's medical record.

The areas usually treated are:

- Face
- Neck
- Cleavage

RECOMMENDATIONS FOR PATIENTS

After the treatment and during 12 hours: avoid extreme temperatures, saunas and Hammam, direct exposure to sun or UV. Avoid to make-up or apply on skin products other than those advised by the doctor. The physician must inform the patient of possible local reactions related to the implantation of this resorbable device: redness and/or local inflammation which disappears within 12-24 hours. Slight oedema or small ecchymoses may occur, which usually disappear within 24 or 48 hours. Should any inflammatory reaction or any other side effect has not disappeared within a week, the patient must inform the clinician about it immediately. The doctor will treat the patient via appropriate treatment adapted to the problem.

The doctor is requested to report any notable side effect related to the **RRS® HA SKIN RELAX WITH BONTA 568®** to **Skin Tech Pharma Group SL**. through the website www.skintechpharmagroup.com, by e-mail at info@skintechpharmagroup.com or by post.

CONTRAINDICATIONS

Allergy to any of the ingredients. Patients presenting any skin alteration, skin disease, infections or sequelae of streptococcal infections. Patients taking immunosuppressants, undergoing cortical therapy, with autoimmune disease history, patients with uncompensated diabetes, acute joint rheumatics, repetitive angina, and endocarditis. No studies are available for use during

pregnancy and breastfeeding or in case of treatment on children or minors under 18.

PRECAUTIONS

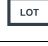
In case of allergic risk, a test should be done before the first injection. The test results must be attached to the patient's medical record. Do not re-use vial once opened. Do not use if the external protecting package has been seriously damaged or if you see some liquid inside of the box. Do not re-use or re-sterilise the unused leftover product; throw the leftover product away; throw the needle and the vial into a specific containers.

Inform the patients under anticoagulant treatment about the risk of ecchymoses or hematomas due to injections. The patient should avoid taking acetylsalicylic acid or anti-inflammatory drug during the week preceding injections. Avoid direct contact with eyes. Do not inject into blood vessels, bones, tendons, muscles, ligaments or any organ different of skin. Prior to injecting, check the expiry date of vials, syringes and needles; also check the integrity of the vial seal. Do not use a product which is expired or looks damaged. Caution: the outside of the presentation is not sterile. Once the presentation has been opened, use immediately. Do not heat, do not frost the presentation. Inform the patient of product characteristics and possible side effects, precautions and contraindications.

There is no study available regarding cross tolerance or possible reaction between **RRS® HA SKIN RELAX WITH BONTA 568®** and definitive filler products, even if it is not injected in the same plane or area. Should definitive implants exist, **RRS® HA SKIN RELAX WITH BONTA 568®** may be used transdermally (protocol with **AD ROLL TD®** or **ExcellDerm® Pro**, without injecting.) Hyaluronic acid precipitates when coming into contact with quaternary ammonium cations (such as benzalkonium chloride), therefore we do not recommend **RRS® HA SKIN RELAX WITH BONTA 568®** coming into contact with said product.

STORAGE

Keep away from the light, under the temperature between 2 and 35 °C. Do not freeze, do not heat.

	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C - 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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Držite podalje od svjetla, na temperaturi između 2 i 35 ° C.

Ne zamrzavajte, ne zagrijavajte.

	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Do not use if package is damaged
	Do not reuse
	Do not restirilize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Do not use if package is damaged
	Do not reuse
	Do not restirize
	Sterilized using steam or dry heat
	Temperature limitation
	Does not contain latex
	Keep away from sunlight
	Keep dry

CE	Notified body number
----	----------------------

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	Manufacturer / Fabricant / Fabricante: SKIN TECH PHARMA GROUP, S.L. C/ Pla de l'Estany, 29 17486 Castelló d'Empúries · Girona (SPAIN) skintechpharmagroup.com info@skintechpharmagroup.com
	Date of manufacture
	Batch code
	Do not use if package is damaged
	Do not reuse
	Do not re-sterilize
	Sterilized using steam or dry heat
	Temperature limitation 2 °C — 35 °C
	Does not contain latex
	Keep away from sunlight
	Keep dry
	Notified body number

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